

Electronic Floor Stand

User Manual



Specification

Size	1700*800*600mm
Gross weight	31kg
Net weight	26.5kg
Running height	70cm-160cm
Input voltage	100-240V-AC
Output voltage	29V-DC
Standby power	< 0.1W
Warranty	1 year
Battery type	23A-12V (remote control)
Wire control	preset 4 different heights

Component Parts




A) Square Post
(1)



D) Base
(1)



E) Universal Wheel
(4)

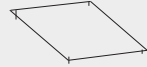

E) Base Plate
(4)




H) Remote Control
(1)



I) Power Cord
(1)



J) Adaptor
(1)

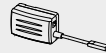

K) Control Box
(1)



L) Wire Controller
(1)



M) Fixing Rod For Lamp
(2)



N) Transverse Brace
(1)



H) Lifting Rod Connection
(1)

Irradiance



M) Screws
M6*14(12)



N) Screws
M8*65(2)



O) 4mmHexagon wrench
(1)



P) 5mmHexagon wrench
(1)



P) 6mmHexagon wrench
(1)




R) Screws
M8*20(4)





S) Cross Screws
M4*8(6)

1. Put the control system into the corresponding position according the picture, except the wire controller.

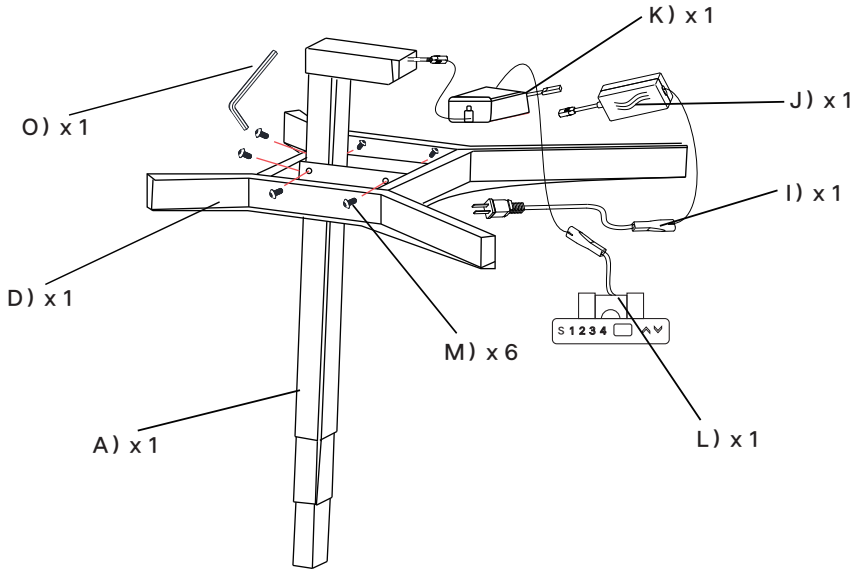


2. Fix cross screw M4*8 by cross screwdriver as picture.

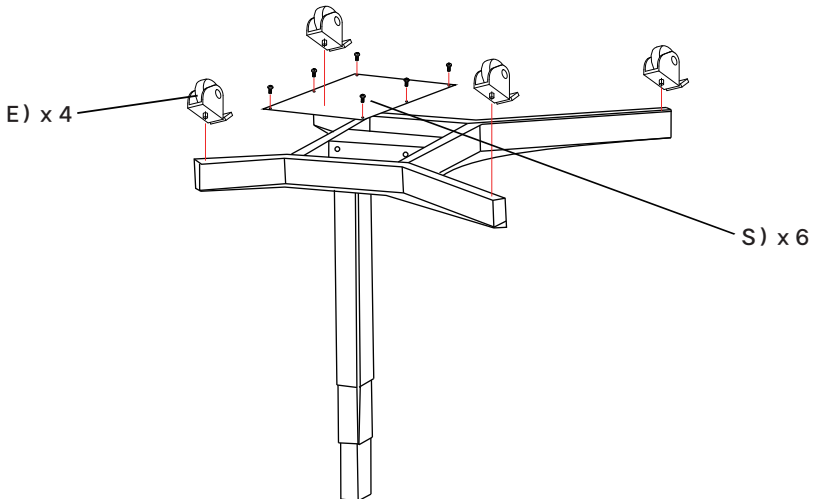

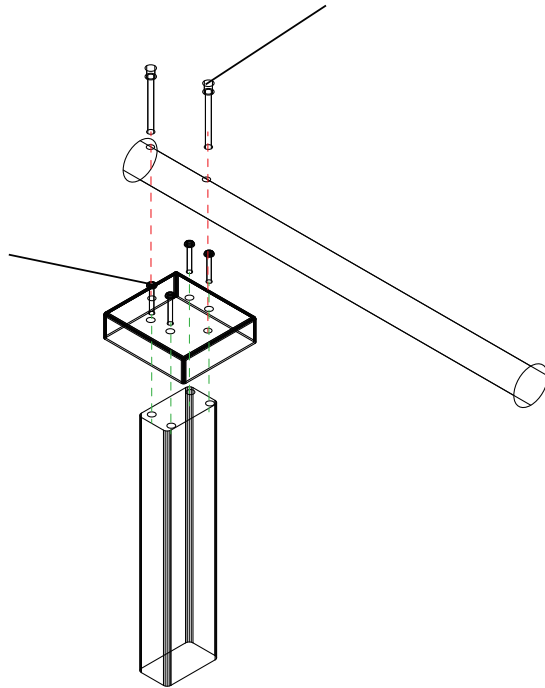


Operation Manual

Remote Control

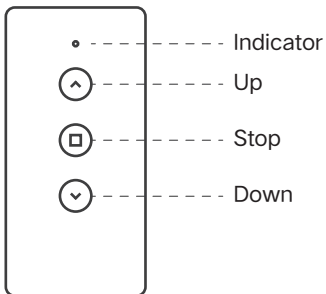

Controller

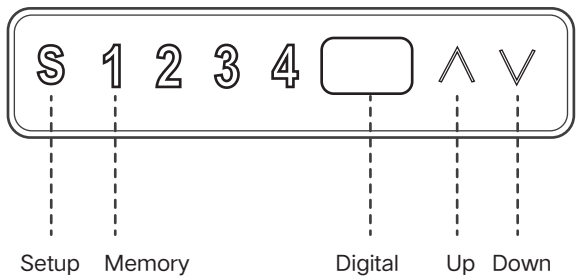

Remote control matching

Keep press the up button " ^ ", also connect both adapt and controller box meanwhile,until we hear the buzzing.

Reset Operation

After successful connection between control system and Lift Column,please hold the Down Button " ~ "for several seconds,the system enters initialization mode. Then remove handset until handset of LED display shows number(height).

Column Stop Operation

Remote Control:When the column during rising, you want to stop it,then you can press the button " o ",the column will stop to rise.And if necessary to stop the column during the descent,you can press the button"o",it will stop to go down.

Memory function

Handset:Touch "S" button,indicator show "S-",meanwhile "S-" flicker,at the same time touch "1-4" button to reserve this position to relative "1-4",four memory positions can reserve.

Unit Conversion

Keep press"S" for 8s,the unit "cm""inch"system will switch automatically.

Operation System Protection

After runs continuously for 3~4 times,control system will be in protection position.

The handset shows "HOT",meanwhile,the remote control and handset will be invalid.




18mins later,automatically connect or directly power off.(power off refers to indicator light of adapter goes out)

Precautions:



- 1) The product shall be used in accordance with the technical parameters;
- 2) Keep away from corrosive gas and dusty environment,pay attention to waterproof.

It can work normally after doing RESET operation when controller show " RST".



Panel mounted on stand

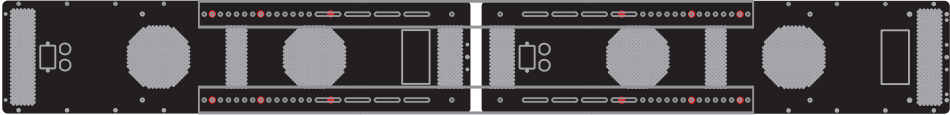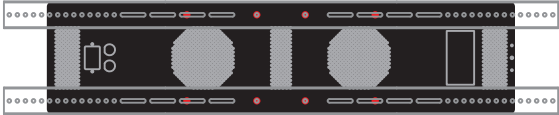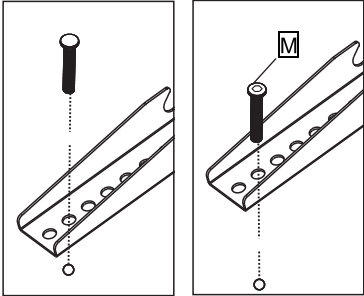


RH1000

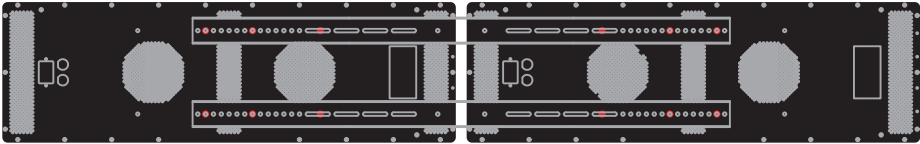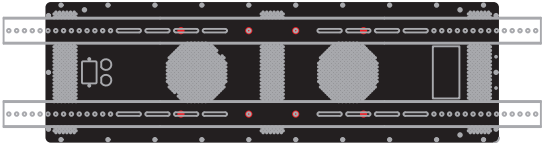

RH1500

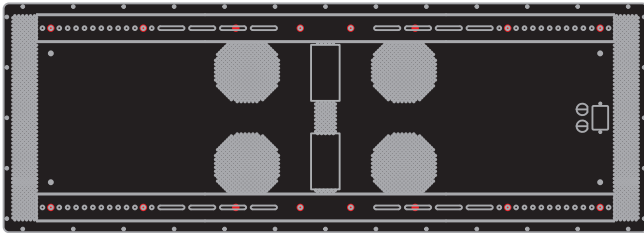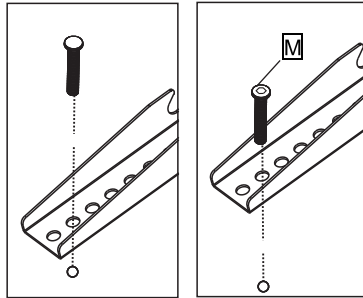


RH2400

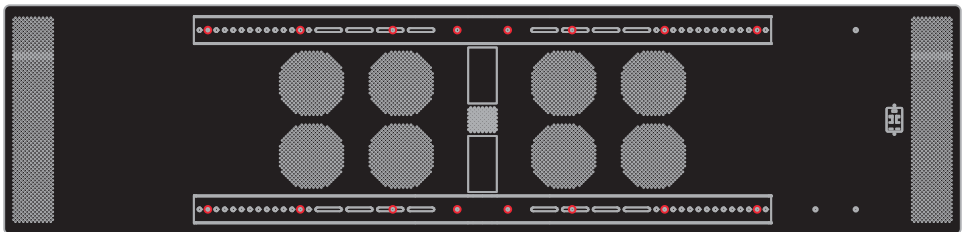

RH3600

Panel mounted on stand

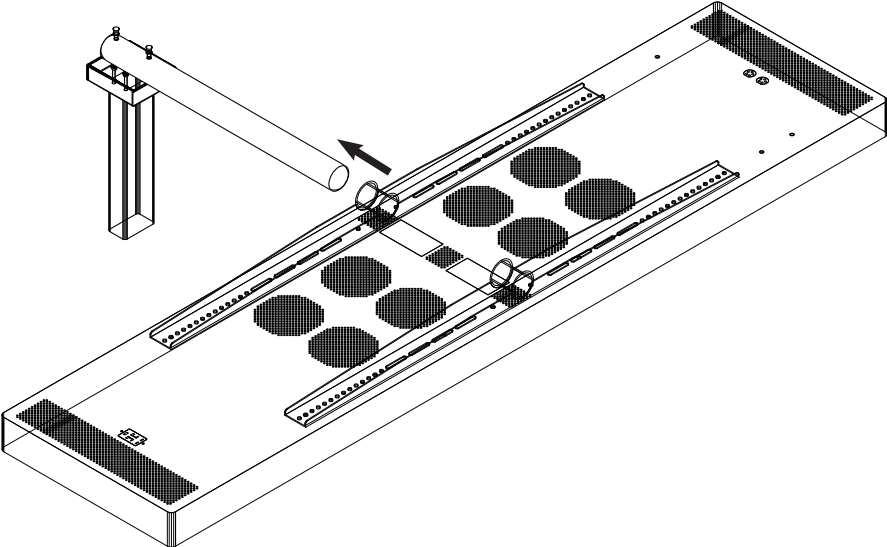

The light panel can be rotated 90° and use for vertical and horizontal use.

