medolight®


by **BIOPTRON®** 
LIGHT THERAPY SYSTEMS By Zepher Group

www.medolight.info

EN	Instruction Manual	4
BG	Инструкция за употреба	9
CS	Použití přístroje medolight	14
DE	Bedienungsanleitung	19
FR	Mode d'emploi	24
HR	Medolight upute za korištenje	29
HU	A készülék ki- és bekapcsolása	34
IT	Manuale d'uso	39
PL	Instrukcja obsługi	44
RO	Manual de utilizare	49
RU	Инструкция по эксплуатации	54
SK	Návod na použitie prístroja medolight	59
SR	Uputstvo za upotrebu	64
UA	Увімкнення та вимкнення апарата	69
AR	تعليمات الاستخدام	74

Using medolight

Medolight is used for light therapy with light emitted by electro-luminescent diodes (LED) in red and infrared wavelengths range.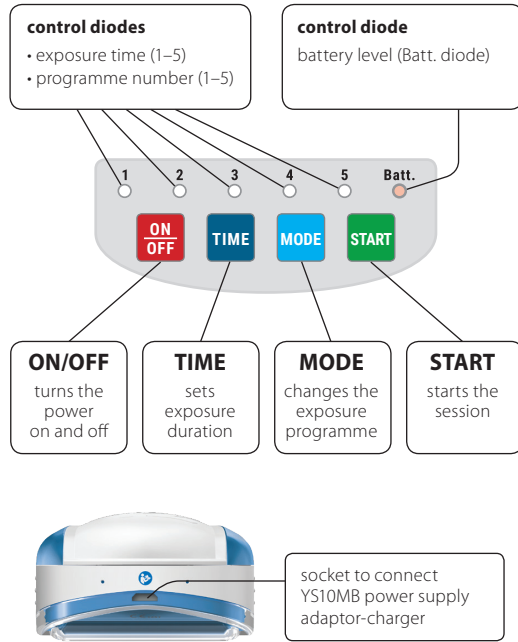

General safety information

During use all the safety precautions which apply to electrical devices must be followed; specifically:

- Please read the manual carefully before operating and follow the instructions therein. It is recommended to complete training with a seller.
- Note: Use of control elements, settings or execution of other procedures than those specified in this manual may result in uncontrolled exposure to light.
- Do not direct light towards your eyes. In case of face exposure close your eyes.
- Treatment area should be exposed from a distance of 5mm.
- Handle the device with care: protect from mechanical damage, fall, moisture and high temperature.
- Do not submerge the device and the charger in water or other liquids.
- The device should not be exposed to direct sunlight.
- Do not use the device or the charger if their casings have been damaged.
- Use only YS10MB charger provided with medolight package. The use of other charger may damage the device. YS10MB charger should be used only with medolight device. The use for other purposes may damage the charger.
- When using the device with the charger exercise particular caution – keep away from children.
- If necessary use an approved extension cord.
- Do not open the casings and do not make repairs yourself. In case of the improper work of the device, repair should be conducted only by an authorized service provider.

WARNING:

- **The device is a source of heat.**
- **The device contains a battery, which may be harmful for the environment – dispose only into a used battery storage container.**

- **Packaging, the device and the charger casings and manual are recyclable.**

Turning the device on and off

Before starting up the device for the first time there should be carried out 3 hours battery charging process - see page 7 - battery status signaling.

The device is turned on and off with the **ON/OFF** button.

After turning on a testing programme is run – **control diodes 1–5** and the **Batt.** diode consecutively lit up.

The device automatically sets the first exposure programme and exposure duration to 5 minutes (**control diode 1 is lit up**).

At the same time the **Batt.** diode is lit up for 5 seconds, indicating the battery level. (see chapter “Battery level indicator”).

If within 1 minute since turning the device on no control button is pressed, the device will switch into the stand-by mode – no control diode is lit up, the device “remembers” the last setting. The device can be switched back into the active mode by pressing one of the three buttons: **TIME, MODE or START**.

Exposure duration selector


Use **TIME** button to select one of five exposure durations: **5, 10, 15, 20, 25 minutes**.

After turning the device on with **ON/OFF** button, the 5 minute exposure duration is selected automatically. Each depression of **TIME** button increases the duration by 5 minutes, causing another control diode to light up for 5 seconds.


After 5 seconds or after pressing the **MODE** button, the device automatically reverts to the selected exposure programme.

Exposure duration selector


5 mins.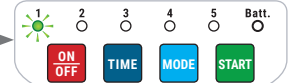


10 mins.

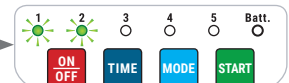
15 mins.

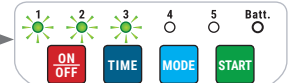20 mins.

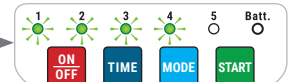25 mins.

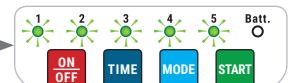Exposure programme selection

Pressing the **MODE** button allows to select one of the five exposure programmes. After turning the device on with the **ON/OFF** button, programme 1 is selected automatically. Each depression of the **MODE** button causes the exposure programme to change, as indicated by the lighting up of the appropriate control diode.

Additional information

- When the device signalizes the selected exposure programme, pressing the **TIME** button allows to check which duration has been selected.
- Setting the “Exposure duration selection” and “exposure programme selection” functions can be done interchangeably and modified until the **START** button is pressed.
- Pressing and holding the **TIME** button causes the exposure duration to change automatically. Similarly, pressing and holding the **MODE** button causes the exposure programme to change automatically.

Starting the session

The exposure session begins after pressing the **START** button.


End of exposure session

The exposure session ends automatically after the selected exposure time ends. After 1 minute after the end of the session the device switches into the stand-by mode. The session can be stopped at any moment by pressing the **ON/OFF** button – turning the device off.

Additional information

- Pressing the **ON/OFF** button erases the selected programme and exposure duration. After turning it on again, the device resets to the initial programme 1 and 5 minute duration.
- During work buttons **TIME**, **MODE**, **START** are inactive.
- A sound signalizes the end of the session.

Exposure programme selection

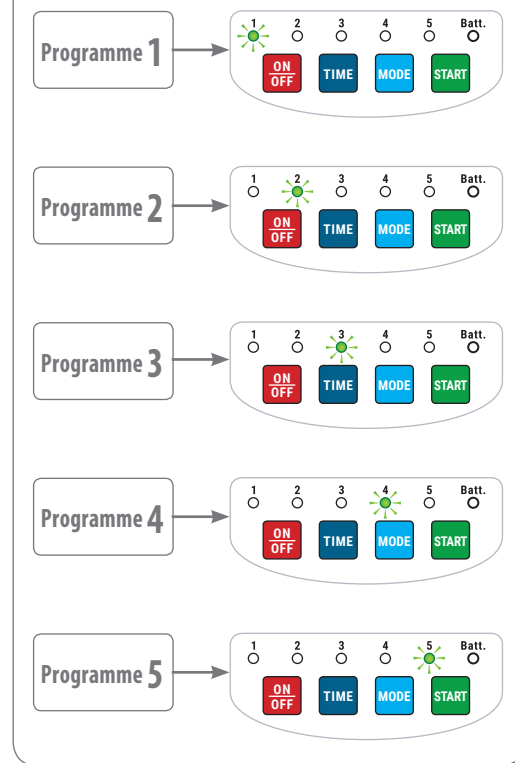- A sound signalizes also if any of the three control buttons: **TIME**, **MODE**, **START** have been pressed.
- If within 1 minute since the end of the session none of the three buttons: **TIME**, **MODE**, **START** has been pressed, the device switches into the stand-by mode, keeping the selected setting in memory. It can be switched back into the active mode, keeping the previous setting, after pressing one of the three buttons: **TIME**, **MODE** or **START**. Pressing the **ON/OFF** button in the stand-by mode will erase the previous setting and automatically set the first exposure programme with exposure time 5 minutes.

WARNING: The device is a source of heat. After each session using programme 1 one should make a break to cool the device off (with 10 mins exposure in programme 1, the break should last about 15 minutes)

Battery level indicator

Before first use of device You should perform three hours (3-hours) battery charging process.

Battery charging

Battery charging follows after connecting the YS10MB charger to the device socket and next to the main power supply.

The full charging of the battery follows after 3 hours of continuous, uninterrupted charging. While charging the battery the device remains active and the charging process is under control.

The battery charge state is indicated by several fold (repeated) sound signal and lighting of the Batt. diode green (until the moment of disconnecting the charger from the power supply).

Leaving the charging for more than 3 hours poses no threat.

The charging can be started in any state of device (switched on or switched off). For performing the correct process of battery forming should be conducted at least (minimum) 3-times battery charging and discharging process.

The Batt. diode indicates the battery level in three phases of work:

1. After turning the device on:

- the Batt. diode lights up green – battery is full, the device can be used without using charger,
- the Batt. diode lights up red – the battery is depleted (it is necessary to use the charger).

2. During work:

- the Batt. diode does not light up – battery is full, the device is ready for use,
- the Batt. diode lights up red at intervals – the battery is partially depleted, however using the device can continue,
- the Batt. diode lights up red constantly – is almost depleted, using the device can continue,
- the Batt. diode lights up red and there is several fold sound signal – the exposure is terminated and device switched off; the battery is discharged. You should connect the charger and perform the 3-hours charging process.

3. During battery recharging:

- the Batt. diode lights up red – the battery is discharged,
- the Batt. diode changes to green, there is sound signal – the battery is charged,
- after disconnecting the charger the Batt. diode goes off.

The device can be used for treatments even during charging, but since this shortens the battery life, we recommend not to do this.

WARNING! If the device has not be used for a long time, we recommend 3-hours battery charging before starting.

PARAMETER	VALUE
Internal Battery	3,7V 1700mAh, Li-ION
Power supply adaptor-charger	YS10MB – normal input voltage 100-240V / 50Hz – normal output voltage 5,4V ±10%
Power Consumption	max 8 W
Operation time of medolight (after the battery is fully charged)	up to 60 minutes
Battery recharging time	approx. 3 hours
Emitted wavelengths	– infrared 880 ± 30 nm – red 640 ± 30 nm
Duration time impulse / pause / repeatability	continuous light – Program 1 0,05s/0,05s/100% – Program 2 8,3-10 ⁻⁴ s/8,3-10 ⁻⁴ s/100% – Program 3 1,6-10 ⁻⁴ s/1,6-10 ⁻⁴ s/100% – Program 4 6,25-10 ⁻² s/6,25-10 ⁻² s/100%
Power density max/min	26mW/cm ² / 18mW/cm ²
Average power deviation	
Light energy in 5 minutes	max 8 J/cm ²
Environmental conditions	– operation from +10°C to +40°C – transport and storage from -20°C to +45°C
Norms met	EN 60601-1 EN 60601-1-2 EN 60601-2-57
Medolight has been approved as a medical device as per EU directive	93/42/EEC
Size	length / width / height 130 / 71 / 42 mm
Mass	– medolight device 200 g – power supply adaptor-charger 75 g

Maintenance

- If it is necessary to disinfect the device, including the surface in contact with the body, use a cloth dipped in a fluid designed for the disinfection of medical devices available in pharmacies or shops selling medical equipment (for example 96% ethyl alcohol or chlorhexidine).
- The cleaning and disinfection of the device and charger should be done after disconnecting them from the power source.
- Do not submerge the device or charger in water and other liquids.

Explanation of symbols used

	Device Class II		BF Type applied part
	Note, follow the manual for user		It is forbidden to put the used equipment including other waste
	Use indoors		Symbol: MANUFACTURER
	Product is in compliance with the essential concerning its requirements of EU directives for medical devices		
	Eurasian Conformity mark		

Technical data related to electromagnetic compatibility can be found on www.medolight.info

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